

Omnigen® : Disposal Information

Omnigen® should be **discarded in a clinical waste bin**, following your centres standard operating procedures. Residue of Omnigen® may remain in OmniLenz® when removed from the patient, thus, OmniLenz® should be discarded in the same way .

If the unit has not been used for human application, the disposal of the unit should be noted in your centres transplant book for stating the date, reason and method of disposal for traceability purposes .

If you do not have a storage licence for human tissue you are not allowed to keep an Omnigen® unit for more than 48 hours and should discarded the unit.

If any serious adverse events or reactions occur regarding Omnigen®, please report this to NuVision Biotherapies without delay, by phone or email.



For enquiries and further information,
please contact:

NuVision Biotherapies Limited
MediCity, D6 Building
Thane Road
Nottingham
NG90 6BH

✉ info@nu-vision.co.uk


☎ +44 (0)115 784 0120

STERILE	A
---------	---

Omnigen®

Tereo® processed human (derived) amniotic membrane

Storage Guide (United Kingdom)

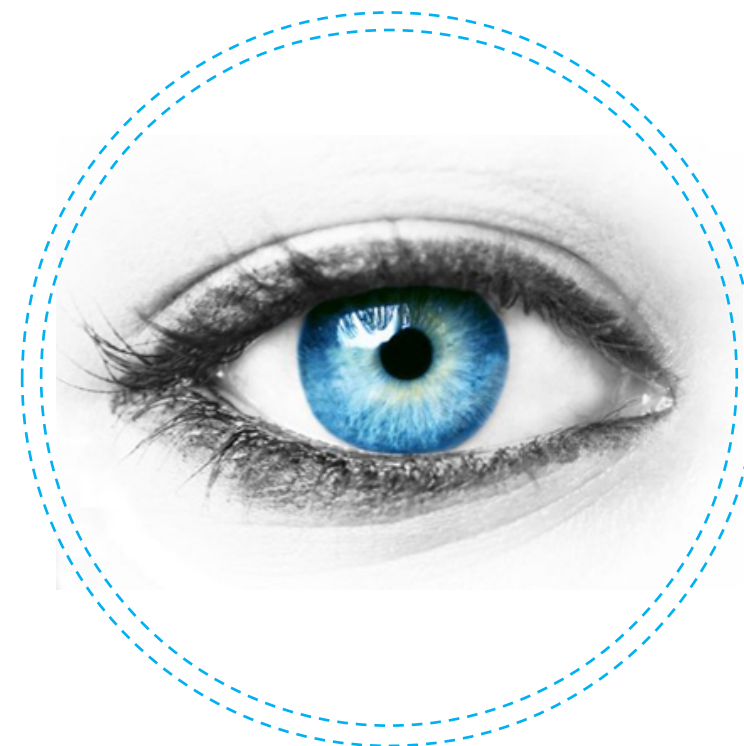

Omnigen storage guide Dec-2020 V01

