



Tereo® processed human
amniotic membrane

INFORMATION CARD

Surgical Application

Graft


(epithelial-side-up)



Logo marker

Patch

(epithelial-side-down)

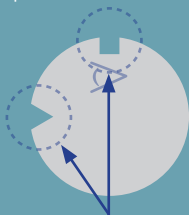

Indentation markers

Your OmniGen® disc will have one or both of the orientation markers (logo or indentations – see above). **Always handle OmniGen with dry instruments. OmniGen can be rehydrated *in situ*.**

OmniLenz® Application



Orientate, centre and load the OmniGen disc. The OmniGen is placed **epithelial-side-up** when loading the OmniLenz.



SAEARs
form



ROMEO™
Ophthalmology



OmniLenz
Application Guide



NuVision Biotherapies
info@nu-vision.co.uk
www.nu-vision.co.uk